ViXion2

ViXion _

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Insert the included USB Type-C cable into the main unit's charging port (A). Charging is complete when the power indicator lights up blue.

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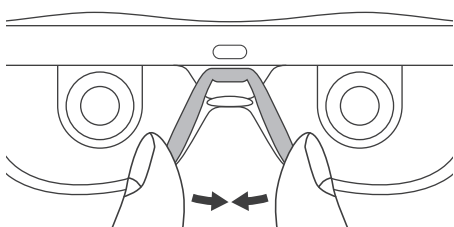
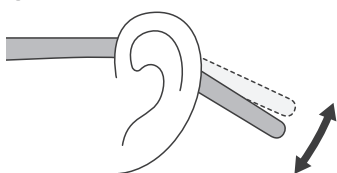
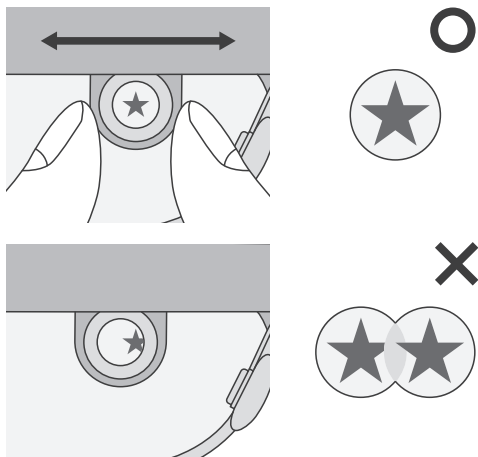
Gently bend the nose pads inward, outward, or sideways to adjust the fit.

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Gradually bend the temples to fit the shape of your ears. To avoid damaging the shape-memory material, do not bend them excessively.

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Put on ViXion2 and close one eye. Looking at a target approximately 1m away, slide the focus lens left or right to center the target within the black frame. Repeat for the other eye. Position adjustment is complete when the circular fields of view overlap into a single circle, as shown in the illustration.

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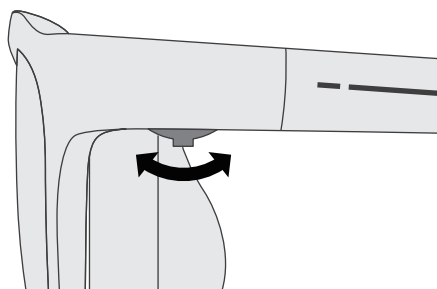
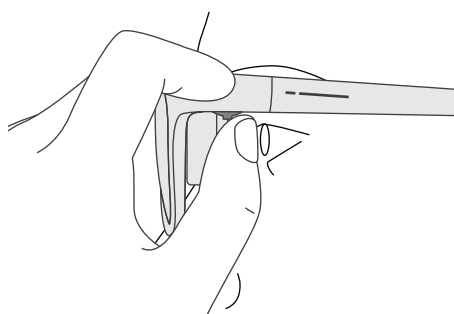
Close one eye and look at a target about 1m away. Keep tilting the lever forward or backward until the image is in clear focus. You will hear a beep while the lever is in use. Repeat for the other eye to complete the adjustment. Releasing the lever activates auto-focus mode and saves your settings for future use.

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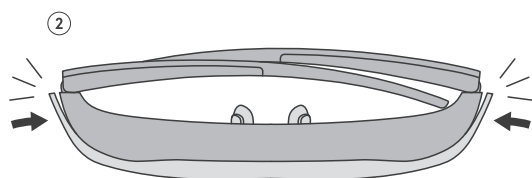
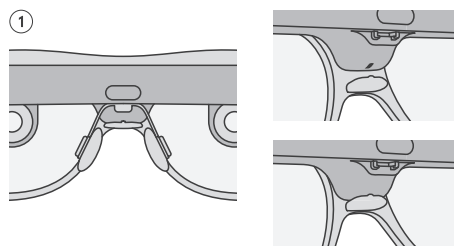
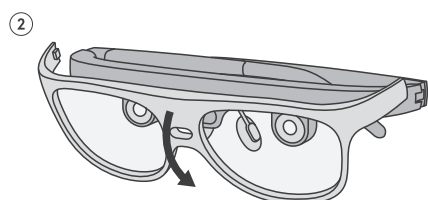
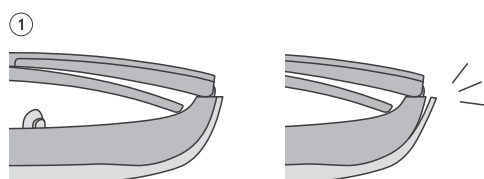
Fold the temples, then pinch the edge of the outer frame and pull it away from the main unit. Repeat on the other side.

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Tilting the outer frame downward, as shown in the illustration, makes it easier to remove.

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Align the protrusion at the bottom of the outer frame with the recess below the nose pads. Once centered, press the top of the outer frame straight into the main unit.

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Press each edge of the outer frame into the main unit until it clicks into place.

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Cautions

- ・ Do not drive a car, ride a bicycle, or operate any vehicle while wearing this product.
- ・ Do not use in situations involving fast movement or sports.
- ・ Do not use the product while it is charging.
- ・ Not waterproof. Do not splash with water or immerse in liquid.
- ・ Avoid using or storing in environments exceeding 50°C (122°F).
- ・ Remove immediately if you notice any abnormality with the product.
- ・ Stop use immediately if you feel unwell.
- ・ This product is not a medical device.

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